




The **New Rhythm** in Sleep

A SLEEP APNEA THERAPY DEVICE— for anytime and anywhere



- ✓ **FDA-cleared** for treating mild to moderate obstructive sleep apnea (OSA)
- ✓ No machine, hose, electricity or batteries needed
- ✓ Use at home, traveling, camping... *use anytime, anywhere*

- ✓ Small enough to fit in a shirt pocket
- ✓ **Reusable** and easy to clean with simply soap and water
- ✓ Designed, molded and assembled in the USA 

Q1. What types of patients may benefit from Bongo Rx?

Patients with mild to moderate OSA, for example those who are:

- Non-compliant with CPAP and looking for a user-friendly option
- Compliant with CPAP, but are looking for a supplemental therapy option for use while traveling, camping, at home, or any place electricity may not be available.
- Newly diagnosed mild to moderate OSA patients without significant co-morbidities



Q2. How does Bongo Rx work?

The Bongo Rx has soft nasal seals that seal within the nasal openings. During inhalation, small valves open to allow you to breathe normally through the device. During exhalation, the valves close directing exhaled air through specifically designed vent holes to generate EPAP (Expiratory Positive Airway Pressure), which keeps open the airway until you inhale again.

FAQs
continued on page 2



888.925.2526 | www.airavant.com
info@airavant.com



The New Rhythm in Sleep

FAQs continued from page 1

Q3. Has the Bongo Rx been clinically tested?

Yes, clinical testing was conducted to demonstrate the safety and effectiveness of the Bongo Rx and testing was reviewed by the FDA. Results demonstrated that the Apnea-Hypopnea Index (AHI) was significantly reduced while using the Bongo Rx compared to the patients' baseline diagnostic. There were no serious adverse events.

Q4. How is Bongo Rx different than Provent?

While both devices produce EPAP, there are several key differences. The Bongo Rx has soft, silicone nasal seals instead of an adhesive seal. It is also reusable. The Bongo Rx acts as a nasal dilator. The Bongo Rx comes in 4 different sizes. The Bongo Rx has both lower inspiratory and expiratory resistance.

Q5. Will I need a prescription for the Bongo Rx?

Yes, as all devices that treat obstructive sleep apnea (OSA) require a prescription in the U.S. Please consult with your physician to see if the Bongo Rx is right for you.

Q6. How do I determine the correct size for the Bongo Rx and where can I purchase it?

The Bongo Rx **All Sizes Kit** is a patient starter kit that includes all 4 sizes, as well as sizing instructions. This All Sizes Kit allows the patient to try all 4 sizes and take their time getting used to the feeling of the product and find the size that fits best. The product can be purchased from several sources. Please ask your physician for referrals to medical equipment suppliers in your area or search the internet for Bongo Rx.

Q7. Is Bongo Rx covered or reimbursed under insurance plans?

Bongo Rx, like many alternative sleep apnea therapy devices, is not a covered item under medical insurance plans. You may be able to utilize your healthcare savings plan for the Bongo Rx. We recommend that you check with your insurance company.



Top 7 Frequently Asked Questions

