

## Drivers Install Instructions

\*\*\*Prior to installing the driver make sure that the CPAP is unplugged from the computer\*\*\*


1. You will only need to do these steps, if the boxes do not become colored when plugging in your CPAP
2. Click [HERE](#) to download the drivers
3. Go to your downloads folder and double click on the file

 CDM v2.12.36.4 W....zip

4. Click Extract All




5. Leave the default folder and click extract

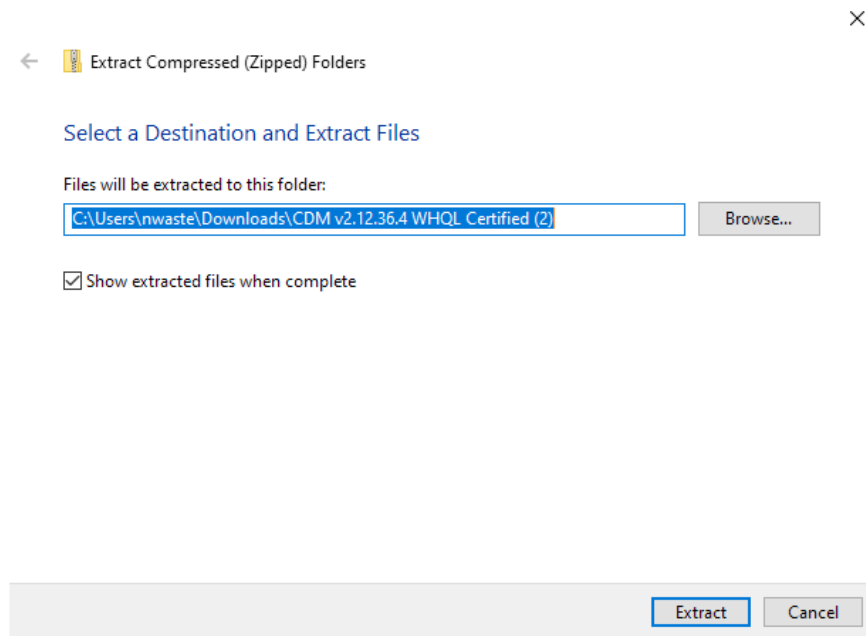


- Find the file called ftdibus, there are 2 of them. One will have an icon of a piece of paper and gear. Right click on it and choose Install from the menu.




- You will get a pop up that the operation completed successfully.

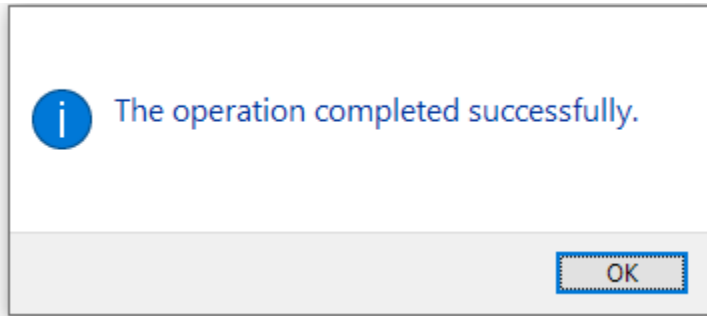

- Find the file called ftdiport, there are 2 of them. One will have an icon of a piece of paper and gear. Right click on it and choose Install from the menu.

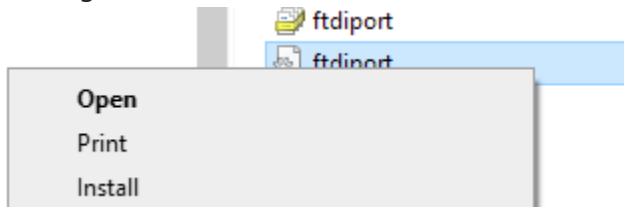

- You will get a pop up that the operation completed successfully.

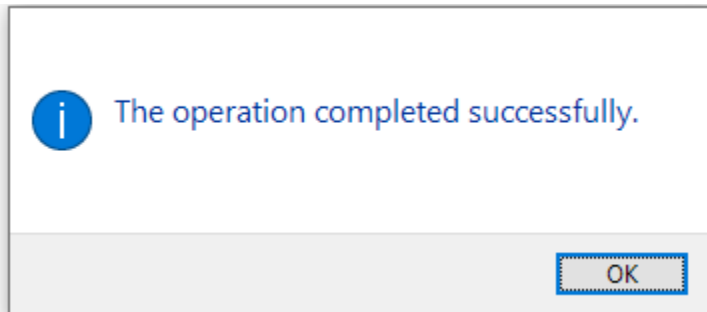

- Reset the computer, open the MiniCPAP software and plug in the device